PATHFAST™ SAMPLE DILUENT 2

[English]

Issue Date: 01/11/2023

Revision: ver.1

Intended use

PATHFAST SAMPLE DILUENT 2 is an in vitro diagnostic product for PATHFAST assay reagents.

PATHFAST SAMPLE DILUENT 2 is used to dilute samples when assay values exceed the reportable assay range on the PATHFAST automated analyser using PATHFAST reagents.

PATHFAST SAMPLE DILUENT 2 is intended to be used by laboratory technicians in clinical laboratories.

Package composition of provided materials

PATHFAST SAMPLE DILUENT 2 20 mL x 2 bottles (liquid, Na azide < 0.1%)
Instructions for use 1 sheet

Precautions and warnings

1. Azide can react with the copper and lead used in some plumbing systems to form explosive salts. When disposing of azide-containing materials, they should be flushed away with large volumes of water.
2. Dispose of all measured reagents and materials according to the standard disposal method; for example, autoclave at 121 °C for 20 minutes. Follow general precautions, and handle all components as if they are capable of transmitting infectious agents.
3. This product should not be used for any other purpose.
4. If any serious incident occurs in relation to the product, report it to the manufacturer and to competent authorities in the area in which the user and/or the patient is located.
5. Improper dilution may affect the measured value. Check the accuracy of the measuring instrument.
6. Avoid contamination.

Preparation and procedure

Refer to the each PATHFAST reagent instructions for use for detailed information about dilution.

Reagent preparation

Ready to use.

Storage and expiration

1. Store at 2 - 8 °C.
2. Avoid water damage during storage.
3. The expiration date is listed on each bottle and package.
4. After opening, this reagent can be used until the expiration date.
5. Do not use reagents beyond the indicated expiration date.

Contact for technical assistance

www.pathfast.eu/contact

Symbols

PHC Corporation uses the symbols and signs listed in EN ISO 15223-1:2021 (Medical devices - Symbols to be used with information to be supplied by the manufacturer - Part 1: General requirements).

* PATHFAST: JP Registered Trademark No. 5982733

* PATHFAST: US Registered Trademark No. 3074207

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PHC Europe B.V.